

QRS-Card™ Cardiology Suite 4.06 Software Installation for Windows 7

The first time you use the system, you need to install the **QRS-Card™ Cardiology Suite** software by using the Setup file on the **QRS-Card™ Cardiology Suite** software CD. Once the software is installed, you will be able to use the **QRS-Card™ Cardiology Suite** for Resting, Stress, Holter, Ambulatory BP and Spirometry. It will take approximately 3 to 5 minutes for all of the directories to be created and for all of the files to be copied to your computer hard disk.


1. Insert QRS-Card™ Cardiology Suite 4.05 CD into computer CD-ROM

****Do not connect QRS Card at this time****


2. Click 'Next'




3. Select 'I Accept the License Agreement' and click 'Next'

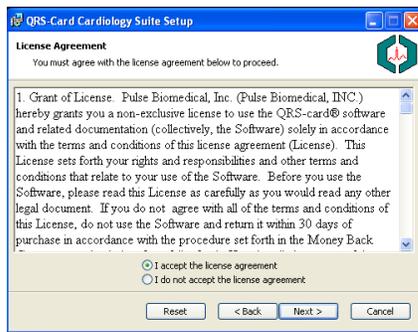


4. Enter Your Name and the Company Name and Click 'Next'

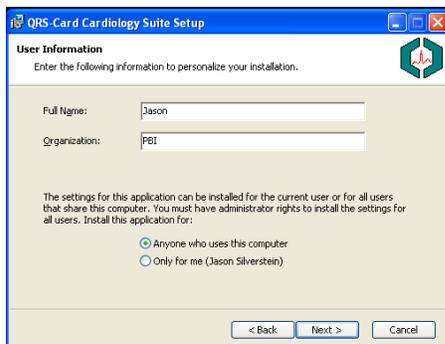


5. Click 'Next'

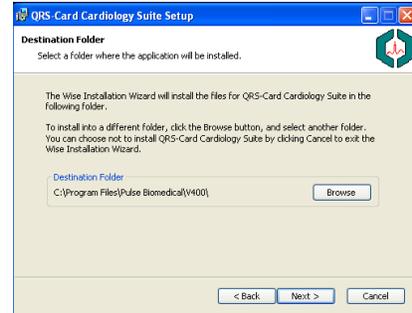

6. Click 'Next'

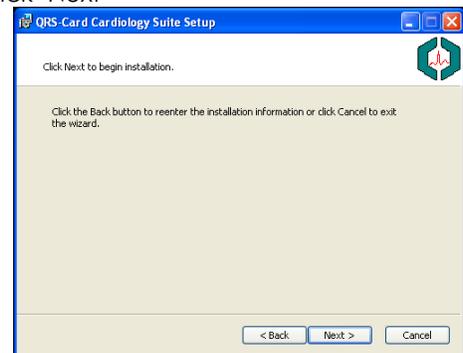

7. Click 'Finish'

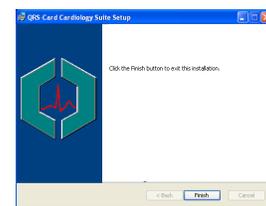

8. Double click QRS-Card Suite shortcut on your desktop.


9. Product ID Window appears, enter the PID from the back of the USB Configuration Key




Or located on this sticker:

Before you click OK. Insert USB Key into computer, Make sure 'NEW hardware has been detected' pops up in the lower right corner of monitor, click 'OK'.


10. The 'Multi task warning box' will appear, click 'OK'
11. Close the QRS Card Suite program.
12. Plug the QRS Card into the USB cable and connect the USB cable to the computer.

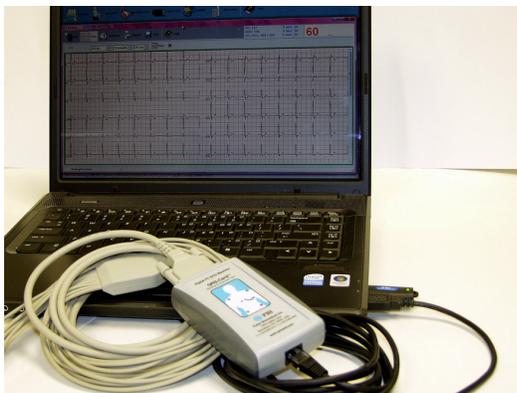

13. The Found New Hardware Wizard box will appear.
 - a. If it doesn't appear, take these steps:
 - i. Go to your Start Menu and right-click on 'Computer.' Select 'Properties'
 - ii. On the "System Properties" screen, click on 'Device Manager' on the left.
 - iii. In the Device Manager, right-click on "QRS Card ECG Recorder" and select 'Update Driver Software'
 - b. Click 'Browse my computer for driver software'

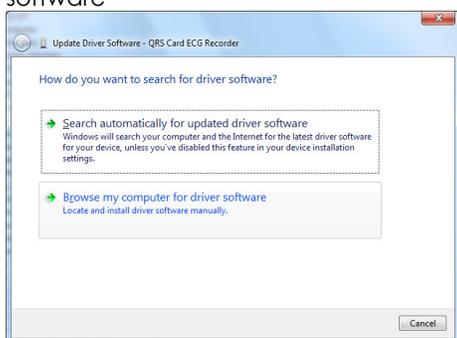

- c. In box under 'Search for driver software in this location,' click Browse and select path: **C:\Program Files\Pulse Biomedical\V400\ftdipbi** (Program Files (x86) if 64-bit OS)

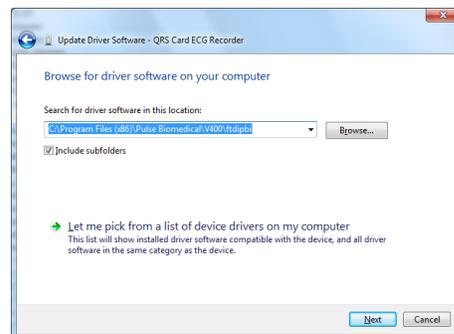

- d. Click 'Next'
- e. Click 'Install this driver software anyway'



- f. Click 'Close'

14. Open QRS Card Suite Program:
 - a. Unselect: 'Show this message again', click 'OK'.
 - i. All this message is saying is DO NOT run any other programs while running test with PBI software.
 - b. Click Rest ECG tab.
 - c. Confirm recorder says QRS Card USB
 - d. Click 'OK'
 - e. You should see artifact/noise scrolling across the screen, system is now functional.

If you purchased a stress system, Click Stress ECG Tab and repeat steps 15C, 15D, 15E

Part: 681101