




# PBI

## Quick Reference Guide

### QRS-Card™ Holter ECG System



## Running a New Holter ECG Test

1. **Prep the patient and apply electrodes** (see Holter hardware user guide for electrode placement). \*A good patient prep is most beneficial for reducing artifact and noise on the ECG
  - a. **Shave hair** if necessary from electrode sites.
  - b. **Rub skin with alcohol** for better electrode contact with skin.
  - c. **Let dry fully.**
2. **Instruct the patient to go about their normal activities**, except for activities such as taking a shower, swimming, or any activity which would cause the electrodes to become loose or fall off.
3. When the patient returns, **remove electrodes and Holter monitor**. Use appropriate equipment to download the Holter ECG data.
4. **Select the Holter ECG Icon**  **OR use hotkey F5.**
5. Once you navigate through the download screen, click 'OK' and the **Holter data will start downloading** to Cardiology Suite software.
6. **Enter patient demographics and click 'Ok'** to complete the download.

## Reviewing a Holter ECG Test

1. To view (or edit) a Holter ECG test, **highlight the desired test from the Cardiology Suite**

**Master Screen test database and click**



**OR double click on the test** to be reviewed.

2. The test record will be displayed in the Cardiology Suite Holter Dataview screen.
3. You can now edit data and print reports.



- Used to view or edit information about the patient and enter diary information.



- Shows various views of the data, including beats and bins, episodes and selected strips.



- provides summarized data and tables for Holter ECG measurements. Also allows user to edit interpretation statements.



- provides access to advanced Holter options.



- allows user to print various ECG reports to printer, PDF or Microsoft® Word.



*The Digital Cardiology Experts*

**Resting, Stress , Holter ECG & Blood Pressure**

*Turn your Notebook, Tablet or Desktop PC into a Medical Device*

Pulse Biomedical, Inc  
112 Ivy Lane, King of Prussia, PA 19406  
Phone: (610) 666-5510  
Fax: (610) 666-5630

Pulse Biomedical GmbH  
Wilhelm Bihler Str 4  
72474 Winterlingen, Germany  
Phone: (49) 7434-316038  
Fax: (49) 7434-316039

[www.QRSCard.com](http://www.QRSCard.com)