

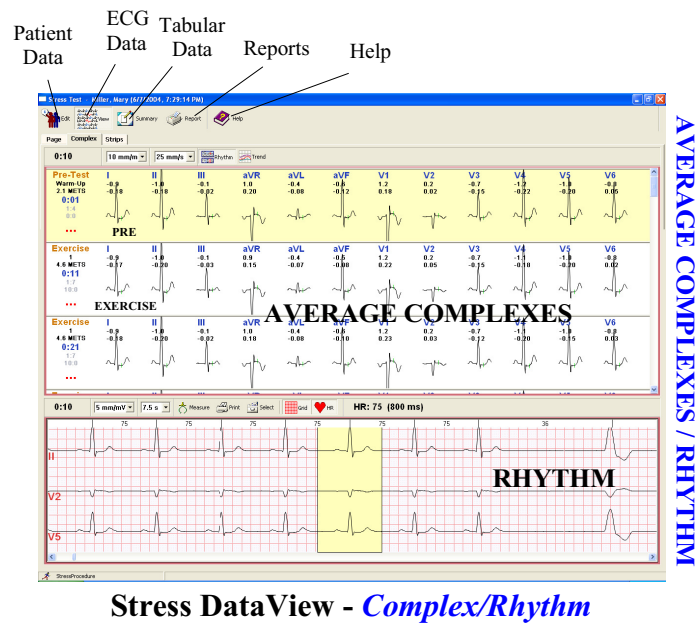
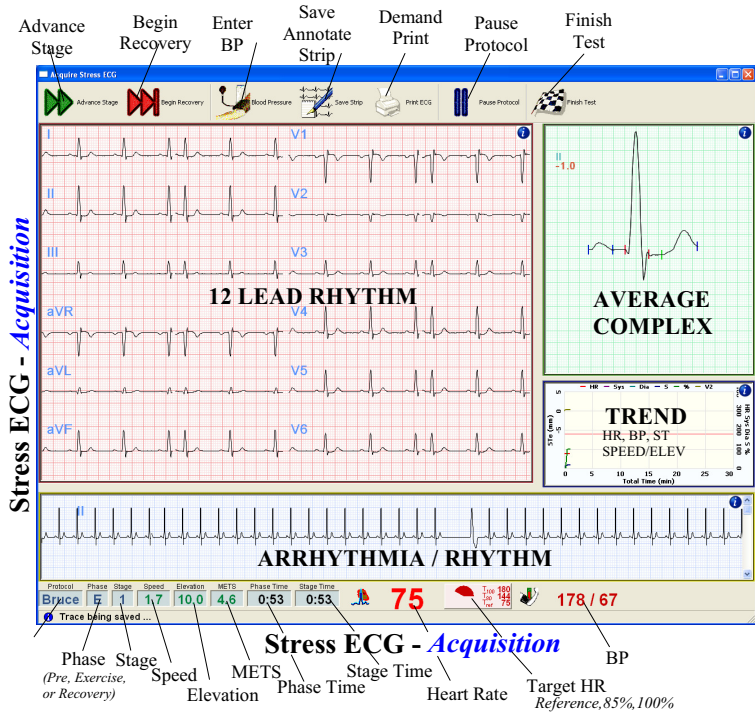
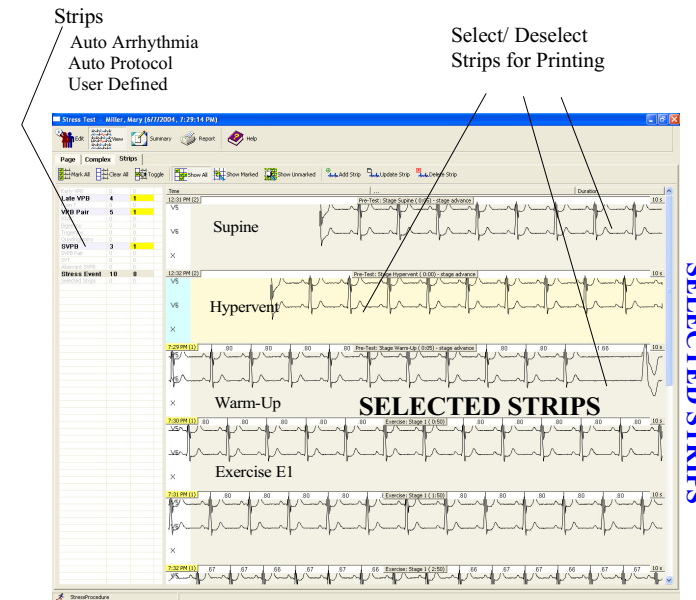


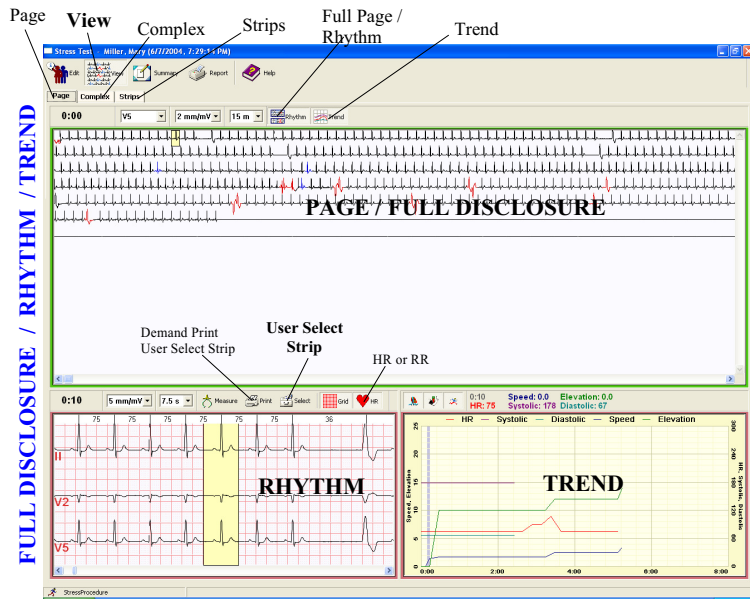

QRS-Card Cardiology Suite 4.0

Resting ECG, Stress ECG, Holter ECG & ABPM

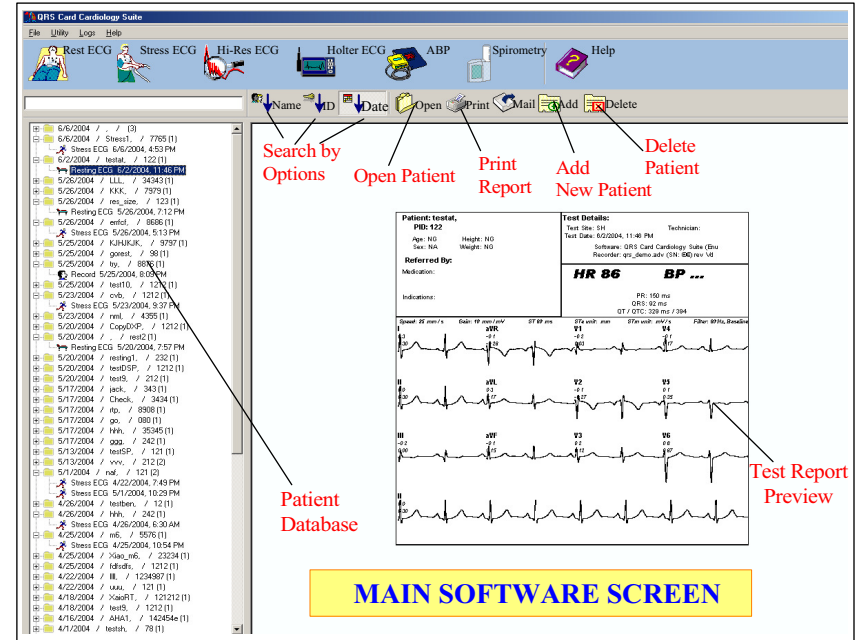

About QRSCard Cardiology Suite 4.0

QRS-Card Cardiology Suite 4.0 is collection of Cardiology Software with single unique user interface and a combined database for Resting ECG, Stress ECG, HiRes ECG, Holter ECG, Ambulatory BP and Spirometry.


QRS-Card Cardiology Suite 4.0 software requires a USB Security Key for performing a real patient test. A demo driver is provided for Resting, Stress and Holter software for tutorial and Software testing purposes. QRS Demo Driver can be selected (full software functionality) under Resting, Stress or Holter Setup under "Recorder" (CTL+F2, CTL+F3, CTL+F3)

The user must setup before the using the software. To setup:

- Click File
- Click Stress
- Click Setup or Press Ctrl + F3 at the same time

DEMO Program / DRIVER does NOT require a PID

- Registration of the software can be performed:
- From Master Screen Press **CTL+F1**
 - To Register software License Press **ALT+R** (USB Security Key & a 17 Digit PID code is required for registration/software activation)

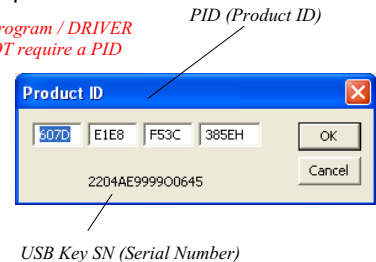

Stress DataView - Page/Rhythm/Trend

Stress DataView - View/Edit Strips



PBI

Pulse Biomedical, Inc.
1305 Catfish Lane, Norristown, PA 19403, USA
Tel: 610-666-5510 Fax: 610-666-5630

Pulse Biomedical GmbH
Wilhelm Bihler Str 4, 72474 Winterlingen, GERMANY
Tel: +49-7434-316038 Fax: +49-7434-316039